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www.tummzy.com o info@tummzy.com.

NATURAL SELF-HEATING PERIOD PATCH.

📖 Please read the instructions for use before applying the product. Store it for later use.

MEDICAL DEVICE:

The natural heating period patches are:

- Self-heating
- Simple, effective and discrete
- And provide up to 12 hours of long-lasting in-depth heat for the relief of period pains.

DIRECTIONS FOR USE:

- Open the sachet with your hands (do not cut open) and take out the heating patch. Do so only just before intended use, as the patch gets activated by air. The heating patch is activated within one minute after opening and provides in-depth heat for up to 12 hours.
- Carefully remove the protective layer from the back of the heating patch.
- **Apply the heating patch with the adhesive side on clothing or underwear over the area in pain. Do not apply directly on the skin!**

⚠️ WARNINGS:

- Never apply the heating patch directly onto the skin. Only intended for external use in the lower abdominal and back region. Do not apply the heating patch on very thin material (for example lace underwear, stockings or silk), since the heating patch could come into contact with the skin.
- The heating patch can cause burns or blisters, if not used correctly – even when emitting less heat.
- Do not use with other heating devices.
- Do not apply additional pressure onto the heating patch (for example laying on it or using it under a tight belt or tight clothing) and leave enough room for unrestricted airflow.
- Pay attention when using the patch and remove the product immediately, if the temperature feels uncomfortable or if you develop a skin irritation.
- Do not use the heating patch if the product was damaged.
- Individuals who cannot remove the patch without assistance (e.g. children, physically disabled people or the elderly) are allowed to use the heating patch only while closely monitored.
- Contact a doctor before using the product, if you suffer from diabetes, heart diseases, circulatory disorders, nerve damage, rheumatoid arthritis or if you are pregnant.
- Remove the heating patch immediately, rinse the affected area with water and consult a doctor immediately if the product was swallowed or came into contact with skin or eyes.
- Consult a doctor if you develop persistent or worsening symptoms as this can be a sign of illness.

PLEASE NOTE:

The heating patch is for single and indirect use only (do not apply directly onto the skin) within 12 hours.

⚠️ THE HEATING PATCH MAY NOT:

- Be used by children under 12 years of age.
- Be used for more than 12 hours within 24 hours.
- Be used on regions of the body, where the individual has a sensory disorder and is unable to feel heat.
- Be used on regions of inflammation, contusions, swellings or injured or damaged skin.
- Be used together with creams, ointments, lotions or sprays that can be applied onto the skin.
- Be used on thin or delicate materials since the adhesive could cause damage.
- Be heated in a microwave or by other devices or methods.
- Come into contact with water or other liquids.

INGREDIENTS:

Iron, active charcoal, vermiculite, water.

DISPOSAL:

Dispose of the product in a general waste bin. The heating patch may not be burned.

STORAGE/EXPIRATION DATE:

Store the product in a place that cannot be accessed by children or pets. Store it in a cool and dry place that is protected from sunlight. Please take note of the expiration date printed on the package.

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