ActiMaris®
+ NASAL Spray

Gebrauchsinformation – ActiMaris® NASAL Spray

1. Produktprofil

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Medizinprodukt: Klasse I, CE

ActiMaris®
+ NASAL Spray

Instructions for use – ActiMaris® NASAL Spray

1. Product profile

ActiMaris® NASAL Spray is a hypertonic, basic-ionised seawater solution that contains sea salt (1.2 %), oxychlorite NaOCl (0.04 %) and hypochlorous acid HOCl (0.004 %) and works on a biophysical basis. The sea salt and oxychlorite have a prophylactic effect (in the event of impending inflammatory and infectious processes). ActiMaris® NASAL Spray has a moisturising and cleansing effect on the mucous membrane. At the same time, through its composition of minerals, salts and trace elements, ActiMaris® NASAL Spray supports the physiological environment on the surface of the mucous membrane. Due to the hypertonicity/hyperosmolarity, the solution has a decongestant effect, dissolves the deposits and dilutes the nasal secretions, and by doing so frees the nasal airway. Due to its cleansing effect and its ability to dissolve secretions and flush out the bioburden (bacteria, viruses, etc.), ActiMaris® NASAL Spray improves well-being and alleviates the typical complaints brought about by the indications mentioned above. ActiMaris® NASAL Spray is gentle on the cells, efficient and fast at the same time. The sea salt and oxychlorite are natural preservatives and guarantee the long-term stability of the ActiMaris® NASAL Spray solution.

ActiMaris® NASAL Spray contains:

- no cortisone
- no xylometazoline
- no alcohol
- no antibiotic

2. Fields of application/indications

ActiMaris® NASAL Spray is primarily used as a prophylactic mucous membrane care product and for:

- Cleaning and moistening the mucous membrane and dissolving mucous membrane deposits
- The prophylactic/preventive effect against inflammations and infections of the mucous membrane is based on the creation of a physiological environment on the surface of the mucous membrane (sea salts, minerals, trace elements).
- Effective against colds/runny nose
- Suitable as an adjuvant option (supplement and support) for otherwise necessary therapeutic measures (e.g. for systemic antibiotic therapy), since no interaction is to be expected
- Prophylaxis against resistant germs (MRSA/VRE)
- Prophylaxis against inflammation in the lower respiratory tract (reduces „postnasal drip“)
- Reduction in halitus nasi (reduces bad smells)
- Suitable for daily hygiene and moistening the nasal mucosa

3. Application instructions and dosage

If the head is bent slightly forward, the spray reaches the middle and lower nasal passage. In this way the covered surface area of the mucous membrane is increased and a better effect is achieved.

Adults and children from 2 years:

Recommended dosage of ActiMaris® NASAL Spray: 2 sprays in each nostril 4 – 5 times daily.

Can be used during pregnancy and lactation.

Infants (under 2 years) and newborns:

1 – 2 sprays in each nostril 2 – 3 times daily as needed.

Infants and toddlers should not be placed on their backs, but instead on the side when ActiMaris® NASAL Spray is applied. The head should be lifted slightly. Always consult your paediatrician before using the product for infants and toddlers.

At the recommended dosage, the 20 ml contents (100 µl per spray application) are sufficient for between 7 and 10 days of therapy (approx. 200 spray applications). An excess dosage is not possible when using ActiMaris® NASAL Spray, nor is there any systemic effect. The nasal solution breaks down into water, salt and oxygen after coming into contact with the mucosa. In the same way, forming a habit or dependency on the ActiMaris nasal solution (as with many other nasal sprays) is not to be expected, since it quickly breaks down into water and salt.

4. Tissue tolerance and biocompatibility

ActiMaris® NASAL Spray is suitable for repeated application, as tissue compatibility and biocompatibility have been proven, even with repeated or multiple applications. ActiMaris® NASAL Spray takes effect very quickly in less than a minute. Its application is painless, may burn slightly in rare cases (due to the sea salt) and is generally well tolerated even by allergy sufferers.

5. Side effects and contraindications

So far, no side effects are known.

Contraindication in case of allergy to ingredients (not observed so far).

6. Interactions

When used correctly, no interactions are to be expected – even with necessary systemic antibiotic therapies. A local combination therapy with another endonasal preparation is not necessary or recommended due to its optimal effect in the stated indications. Please consult your doctor for safety reasons, if you want to systematically use other drugs in combination.

7. General safety instructions

Keep out of the reach of children! Please avoid direct contact with coloured fabrics, as the contained sodium hypochlorite can have an oxidising or bleaching effect! Rinse immediately with water after contact with coloured fabrics.

8. Ingredients and technical data

Water, sea salt, sodium hypochlorite.
Clear, colourless, alkaline and hypertonic sea salt solution with characteristic odour; contains 1.2 % sea salt (Sal Maris), 0.04 % NaOCl and 0.004 % HOCl.

9. Storage instructions, shelf life, period of use

ActiMaris® NASAL Spray should be stored at room temperature between +5° and +25° C. Protect the spray from direct sunlight and/or heat.

The shelf life is 3 months. The current expiration date is printed on the packaging.

After opening the spray, the period of use is 3 months.

ActiMaris AG ■ Manufacturer:

ActiMaris AG
Sandgrube 29
9050 Appenzell (SWITZERLAND)
info@actimaris.com
Phone +41 71 505 75 25

LOT: Batch designation / batch number

REF: Order number

△ Attention, observe instructions for use

■ Can be used until

■ Storage instructions (temperature)

■ Protect from direct sunlight

■ Manufacturer

Information status: 05/2021 – V1.3

Medicinal product: Class I, CE

ActiMaris®
+ NASAL Spray

EC REP

Adriatic Health Factory d.o.o.
Nikole Tesla 17
44000 Sisak
CROATIA

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ActiMaris AG Fabricant :