Portable Nebulizer

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- English...1-21
- Français...23-43

Table of Contents

■ Introduction.....	2
■ Intended use.....	3
■ Safety precautions.....	4
■ Classification and explanation of symbols.....	5
■ Package contents and overview.....	6
■ How to operate with batteries.....	8
■ How to operate with adapter (for Model MBPN002).....	10
■ Cleaning and disinfection.....	12
■ Troubleshooting.....	14
■ Specifications.....	15
■ Technical data.....	16
■ Optional parts and accessories.....	21

Introduction

Thank you for choosing our Portable Nebulizers. Our Portable Nebulizer is a handheld device, designed to aerosolize medication for respiratory therapy purpose. Considering about the difference of your needs between traveling and in house, we are pleased to offer two models that helps you to have better experiences and more conveniences during the treatment.

Portable Nebulizer Model No. MB0500300

This device can operate on two AA Alkaline batteries. With battery and compact size, you are able to continue your treatment while you go on traveling. To help save the planet, we also recommend you to buy AA NiMH rechargeable batteries (optional item, not included) instead of disposable batteries.

Portable Nebulizer Model No. MBPN002

This device has extra power option, it can be driven by both two AA batteries and power adapter (optional item, not included). With additional power adapter, helps you to save battery costs in long term.

Our devices are not only brings you conveniences of lightweight, compact and portable, the virtually silent operation and low medication residue are also features of choosing an Active Vibrating Mesh Technology that had been applied on our devices.

As this device is a medical instrument. Please read Instruction Manual before use, and be sure to follow the instructions of a doctor and use the device correctly.

Intended Use

<p>Medical purpose</p>	<p>This device is designed to nebulize the solution into an aerosol for respiratory therapy purpose.</p>
<p>Intended user</p>	<p>Legally certified medical experts, such as doctors, nurses, therapists, or patients under the guidance of qualified medical experts. The user should also be capable of understanding general operation of this device and the content of instruction manual.</p>
<p>Intended patients</p>	<p>This device design for all ages of patients. Except for the patients who are unconscious, not breathing spontaneously or having pulmonary edema.</p>
<p>Recommended operation environment</p>	<p>This device is intended for using in the medical facility, such as hospital, clinic and doctor's office, a room of general household, and open-air environment with a roof. Temperature range: 10°C to 40°C Humidity: 30 to 85 % RH</p>
<p>Precautions for use</p>	<p>Warnings and cautions described in the manual should be observed.</p>

Safety Precautions

READ ALL WARNINGS AND INSTRUCTIONS BEFORE YOU USE THIS DEVICE.

As with any medical device, this product may become unusable due to an electrical outage, battery depletion, or mechanical impaction. We recommend that you have spare batteries and a backup device available to you.

When you use electrical products, always follow basic safety precautions. As with any electrical device take particular care around children.

WARNINGS

- Only use this device for medications prescribed by your doctor.
- The nebulizer is only intended for respiratory therapy and any other application of this device is improper and dangerous. The manufacturer cannot be held liable for any damage caused by improper or incorrect use.
- Do not share your nebulizer with others. It is intended to be used by a single user. If more than one person use it, there is a risk of spreading infectious illness.
- Clean all parts of your nebulizer before use, after each use and after extended storage.
- Adult supervision is required when this device is used by children and individuals who require special assistance.
- Never operate this device if any of the parts are not working properly or have been damaged.
- Be sure the device has been properly cleaned before use to avoid possible contamination.
- Do not plug in or unplug the Adapter from the electrical outlet with wet hands.

CAUTIONS

- Do not attempt to clean mesh with any foreign objects, it may damage the Mesh.
- Avoid dropping the device otherwise it may not function normally.
- Keep the device away from direct sunlight, excessive heat or cold to avoid damaging the batteries.
- Do not attempt to open, repair or modify this device.
- Do not drop any liquids on the main unit or on the adapter, if you drop liquids on them, wipe it off immediately.
- Follow local laws and recycling plans regarding disposal or recycling of components, batteries and packaging.

Classification and Explanation of Symbols


Symbols	Meaning
	Warning / Caution / Note
	Class II equipment per IEC 60601-1
	Type BF equipment per IEC 60601-1
IP22	Protected against foreign objects equal to or greater than 12.5mm in diameter and against drops of water falling at up to 15° from vertical
	Consult instructions for use
	ON / OFF Button
	DC Power source (Direct Current)
	Disposal of Electrical & Electronic Equipment (WEEE): Do not treat this product as household waste.

Package Contents

EN

Check before use


The following items are contained in the package. Please check all parts for visible damage. Replace any damaged parts before you use this device. In the case of missing parts, malfunction or damage, please contact the store where you purchased from or the nearest dealer.

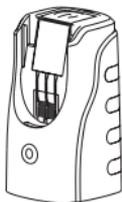1. Main unit

2. Medication cup
(with cover)


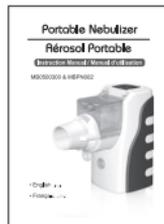
3. Instruction Manual


4. Mask
(Single-use. Do NOT recycle.)

System Overview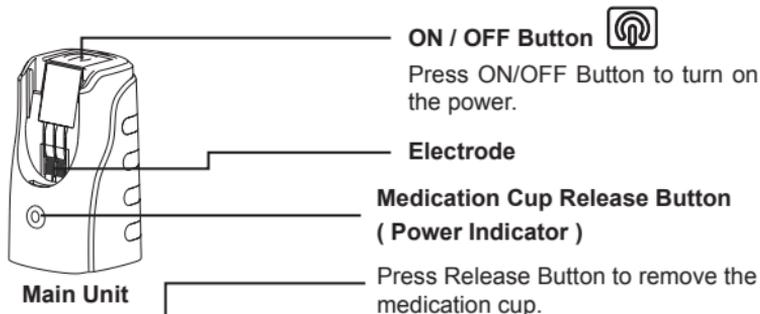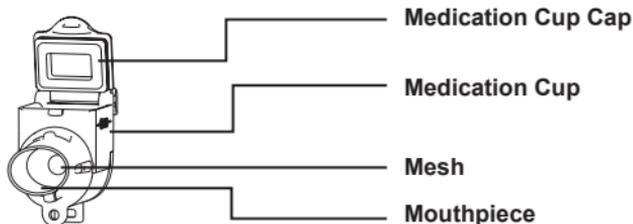


Light indications

- Green light on : Ongoing Operation
- Orange light on/flash : Low power / Malfunction
- No light : Malfunction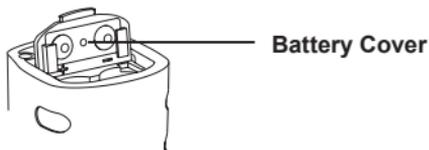

▪ Bottom view (Model no. MBPN002)

▪ Bottom view (Model no. MB0500300)

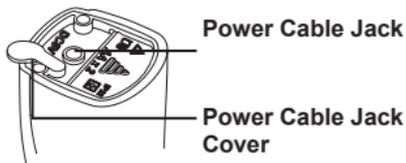
How to operate with Batteries

This device can operate on two AA alkaline batteries or two AA nickel-metal-hydride (NiMH) rechargeable batteries. (Secondary lithium batteries shall comply with the requirements of IEC 62133.)


Clean all parts of your portable nebulizer before use, after each use and after extended storage.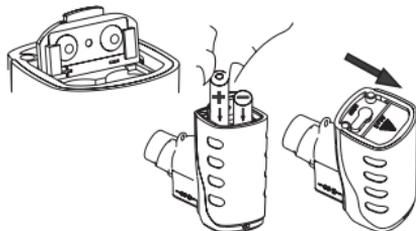

1. Insert the batteries

- Turn the unit upside down.
- Push back and pull up the battery cover to open it.
- Insert two AA alkaline/NiMH rechargeable batteries according to the battery polarity indicated inside the battery compartment.
- Put down and push forward the battery cover to lock it.

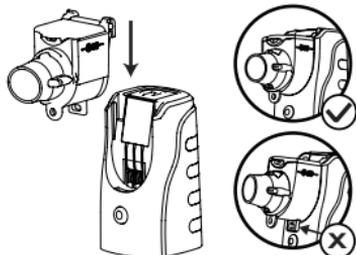2. Set up the medication cup

- Align the three ribs on the back of the medication cup with three trenches of the main unit to insert the medication cup into the main unit.

1. Do NOT insert a battery with the positive(+) and negative(-) poles reversed.
2. Do NOT using old and new batteries together or mixing batteries using differing cell chemistries.

3. Fill the saline

Make sure that indicator light is off before adding the saline.

- Open the medication cup cap.
- Fill the medication cup with saline. **DO NOT EXCEED THE LIMIT (Max. 6ml).**
- Close the medication cup cap properly.

4. Inhalation

- Press ON/OFF button on the top to turn on the device (light Indicator turns on in green).
- Place the mouthpiece between your teeth, with your lips firmly sealed around the mouthpiece.
*If you use a mask, place the mask over your mouth and nose.
- Breathe in and out slowly through your mouth until aerosol formation stop.

5. End of Inhalation

- Press ON/OFF button to turn it off.
- Remove the batteries from the main unit.

1. Tilt the device slightly, while the solution is almost exhausted to make sure that the residual solution contact the mesh and is sprayed completely.
2. If high-viscosity solution is used, nebulization may be reduced.
3. If excessive solution is accumulated on the mesh, nebulization may stop. In this case, turn off the power and absorb the solution by gauze or lint-free towel.


Battery life and battery replacement

1. Battery life is usually depending on the capacity and condition of the batteries. In general, this device can operate about 4 days with two brand new AA alkaline batteries based on usage of 20 minutes a day.
2. Nebulizer can operate on alkaline batteries or nickel-metal-hydride (NiMH) rechargeable batteries.
3. When the power indicator turns into orange (means low power), please replace both batteries with new ones.
4. Battery life are depending on the capacity and condition of the batteries.

How to operate with Adapter (only available on model MBPN002)

1. Only use an external adaptor that is fulfilled in compliance with the requirements of IEC60601-1:2005. Please choose DC 5V/1A adapter, and the connection jack should matches a plug with outer diameter of 3.5mm, interior diameter of 1.35mm, 9.5mm in length and positive polarity .
2. The adapter does not charge batteries. Please remove the batteries before using the adapter.
3. As with all electronic devices, it is recommended that keep the product unplugged when not in use.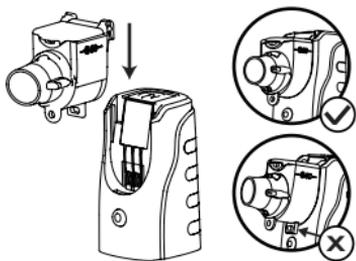


1. Set up the medication cup

- Align the three ribs on the back of the medication cup with three trenches of the main unit to insert the medication cup into the main unit.

2. Fill the saline

- Open the medication cup cap.
- Fill the medication cup with saline.
DO NOT EXCEED THE LIMIT. (Max. 6ml).
- Close the medication cup cap properly.

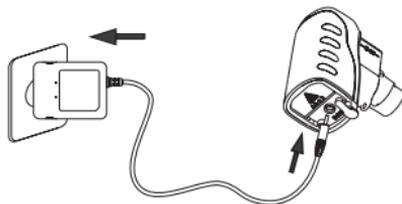3. Connect with adapter

- Plug the cable into the power adapter jack on the main unit.
- Plug the adapter into a standard electrical outlet.

*The power adapter is an optional accessory and its figure might be different.

4. Inhalation

- Press ON/OFF button on the top to turn on the device (light Indicator turns on in green).
- Place the mouthpiece between your teeth, with your lips firmly sealed around the mouthpiece.
- * If you use a mask, place the mask over your mouth and nose.
- Breathe in and out slowly through your mouth until aerosol formation stop.

5. End of Inhalation

- Press ON/OFF button to turn it off.
- Remove the adapter from main unit and electrical outlet.

Cleaning and Disinfection

If the device is not cleaned and disinfected correctly and frequently as indicated, microorganisms may remain in the unit and cause risk of infection.

After every use:

Clean the **medication cup** with distilled water after each inhalation.

1. Pour out the residual solutions in the medication cup.
2. Pour some distilled water into the medication cup.
3. Turn on the device to nebulize the distilled water for 1 to 2 minutes to clean the mesh.
4. Remove batteries or disconnect the adapter from the main unit.
5. Remove the medication cup from the main unit.
6. Wash and rinse the medication cup with distilled water.
7. Shake off excess water and allow parts to be fully air dried on a clean, dry towel.
8. Use gauze or clean towel to wipe off stains on the main unit if necessary.
9. Make sure that all cleaned parts are completely dry before you store them or use them next time.

1. **Keep the battery compartment stay in dry all the time.**
2. **Do NOT poke the mesh with finger, cotton swab or any objects.**
3. **Do NOT clean parts in a dishwasher.**
4. **Do NOT use microwave to dry any parts.**

Daily disinfections

It is important to disinfect the medication cup on a daily basis.

1. Disinfection by boiled water

- Rinse the medication cup with distilled water.
- Bring a saucepan of DISTILLED water to the boil.
- Carefully immerse the medication cup in the boiled water for a maximum ten (10) minutes.
- Carefully to remove the medication cup from the boiled water and shake off excess water.
- Allow parts cooling down and fully air dry on a clean, dry towel, and out of reach of children.
- Make sure that all parts are clean and dry before storage or use.


2. Disinfection by alcohol

- Rinse the medication cup with distilled water.
- Immerse the medication cup in 75% ethyl alcohol for one (1) minute.
- Rinse the medication cup with distilled water again, shake off excess water and allow parts to be fully air dried while lying on a clean dry towel.
- Make sure that all parts are clean and dry before storage or use.


1. Do NOT boil medication cup directly.
2. Alcohol is highly flammable. Do Not use alcohol within the vicinity of open fire or smoke.

The following items are able to be disinfected by boiled water or alcohol.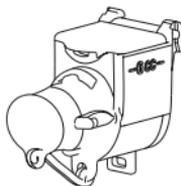


- Medication cup (with cover)

Parts below are NOT able to be disinfected by boiled water or alcohol.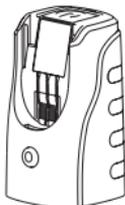

- Main unit

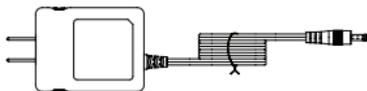- Power adapter* (only available on Model MBPN002)
*The adapter figure might be different from above.

Do NOT rinse or immerse the main unit in any liquid.

Carrying and storing

Put on the mouthpiece cover, store the device and medication cup in a dry and clean environment.

1. Do NOT leave or carry the device containing residual liquid in the medication cup.
2. Remove the batteries if device is not in use for long periods. Failure to do so could result in damage due to battery leakage.
3. Do NOT leave the device under direct sunlight, in high humidity, extreme heat or cold environment.
4. Keep this device away from fire, high electromagnetic fields and out of the reach of children.

Troubleshooting

If any trouble occurs while you are using the device, please check the following list first.

Problem	Possible Cause	Action
Low atomization	Low battery power	Replace the batteries or use the adapter
	The stains on the electrode cause a fault connection	Use rubbing alcohol to clean electrodes
	The mesh holes are clogged	Refer to Cleaning and Disinfection procedure to clean the medication cup
	The mesh is broken	Replace the medication cup
Non-operation even if power switch is on (power indicator does not light and no mist comes out).	The batteries are inserted in the wrong direction	Follow (+) and (-) mark on the battery cover to re-insert the batteries in correct direction
	Batteries are dead	Replace the batteries or use the adapter
	Fault connection between adapter and main unit (only available on Model MBPN002)	Check and reconnect the adapter to main unit
No atomization even if power switch and indicator are on.	The medication cup is not installed properly	Refer to Set up the medication cup procedure to re-install the medication cup
	The mesh holes are clogged	Refer to Cleaning and Disinfection procedure to clean the medication cup
	The mesh is broken	Replace the medication cup
Power indicator turns into orange light constantly.	Low battery power	Replace the batteries or use the adapter
Power indicator turns into orange light and flashing.	Fault connection of electrode and main unit	Use rubbing alcohol to clean electrodes and re-stall the medication cup
	The mesh is broken	Replace the medication cup

- 1. If the device does not nebulize normally after taking the above-mentioned procedure, please contact the store where you purchased the device or the nearest dealer.**
- 2. Never operate this device if any of the parts are not working properly or have been damaged.**
- 3. We recommends to exchange the medication cup after approximately 1 year, and 2 years for the device.**

Specifications

Product	Portable Nebulizer
Model	MBPN002、MB0500300
Method of operation	Active Vibrating Mesh Technology
Power supply	AA Alkaline Battery x 2 AA NiMH rechargeable Battery x 2 Power Adaptor*(only available to MBPN002)
Power consumption	< 1.5W **
Vibrating frequency	Approx. 117 kHz +/- 15%
Nebulization rate	≥ 0.25 ml/min
Particle size	MMAD < 5μm**
Capacity of medication cup	Approx. 6 ml
Dimension	Approx. L76.5 X W41 X H73 mm
Weight	Approx. 74g (Battery excludes)
Operating temperature and humidity	10 ~40°C, 30 ~ 85 % RH, 800~1060hPa
Storage and delivery condition	-20 ~ 70°C, 20 ~ 75 % RH, 800~1060hPa

*** The optional power adaptor is only available on Model MBPN002, and it shall be fulfilled in compliance with the requirements of IEC60601-1:2005. Contact to the nearest dealer for further information. Secondary lithium batteries shall comply with the requirements of IEC 62133.**


**** Test with physiological saline, and under normal temperature of 23°C.**

Technical Data

EN

According to the Standard EN 13544-1:2007+A1:2009 "Respiratory therapy equipments - Part 1: Nebulizing systems and their components", Annex CC.3 using the multistage cascade impactor method to measure the particle size.

Particle Size	MMAD 3.09µm (MMAD = Mass Median Aerodynamic Diameter)
Medication Cup Capacity	6 ml max.
Noise	Less than 50dB (Around 1 meter)

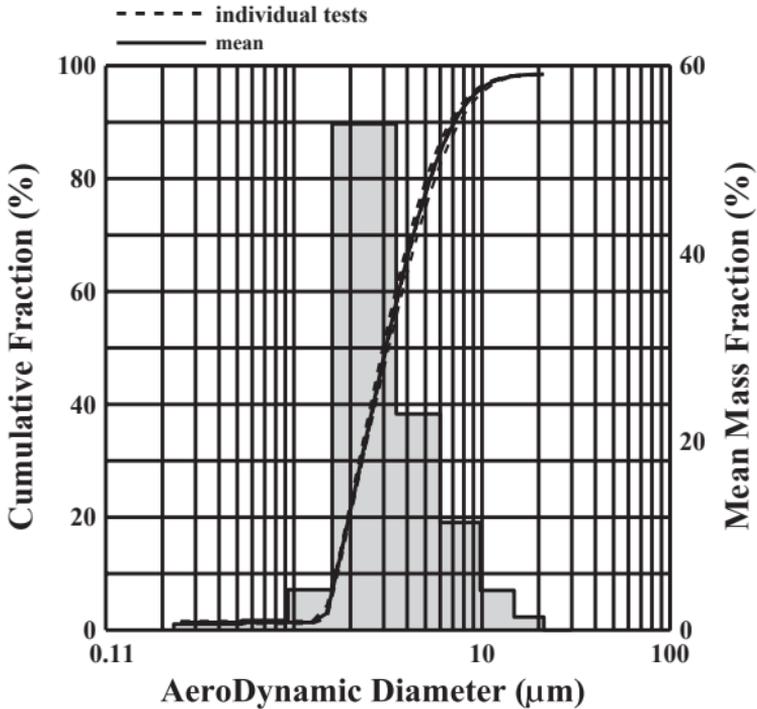Technical Data

In order to regulate the requirements for EMC with the aim to prevent unsafe product situations, the EN60601-1-2 standard has been implemented. The nebulizer MBPN002 and MB0500300 conforms to the EN60601-1-2:2007 standard for both immunity and emissions.

Nevertheless, do not use the nebulizer close to the strong electrical or electromagnetic fields. This may result in incorrect operation and create a potentially unsafe situation.

Guidance and manufacturer's declaration - of MBPN002 and MB0500300

Guidance and manufacturer's declaration-electromagnetic emissions		
<p>The <u>MBPN002 and MB0500300</u> are intended for use in the electromagnetic environment specified below.</p> <p>The customer or the user of the <u>MBPN002 and MB0500300</u> should assure that it is used in such an environment.</p>		
Emission test	Compliance	Electromagnetic environment-guidance
RF emissions CISPR 11	Group 1	The <u>MBPN002 and MB0500300</u> utilisent une énergie RF uniquement pour son fonctionnement interne. Les émissions RF sont faibles et ne peut pas causer d'interférences ua matériel électronique environnant.
RF emissions CISPR 11	Class B	The <u>MBPN002 and MB0500300</u> est adapté pour toute institution, incluant les établissements domestiques et ceux directement connectés au réseau public d'alimentation basse tension qui fournit les habitations pour un usage domestique.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations /flicker emissions IEC 61000-3-3	Compliance	

Technical Data

Guidance and manufacturer's declaration-electromagnetic immunity

The MBPN002 and MB0500300 are intended for use in the electromagnetic environment specified below. The customer or the user of the MBPN002 and MB0500300 should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment-guidance
Electrostatic discharge(ESD) IEC 61000-4-2	+ 6 kV contact + 8 kV air	+ 6 kV contact + 8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/burst IEC 61000-4-4	+ 2kV for power supply lines + 1kV for input/output lines	+ 2kV for power supply lines Not applicable	Main power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	+ 1kV line(s) to line(s) + 2kV line(s) to earth	+ 1kV differential mode Not applicable	Main power quality should be that of a typical commercial or hospital environment.
Voltage Dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5% UT(>95% dip in UT) for 0,5 cycle 40% UT(60% dip in UT) for 5 cycles 70% UT(30% dip in UT) for 25 cycles <5% UT(>95% dip in UT) for 5 sec.	<5% UT(>95% dip in UT) for 0,5 cycle 40% UT(60% dip in UT) for 5 cycles 70% UT(30% dip in UT) for 25 cycles <5% UT(>95% dip in UT) for 5 sec.	Main power quality should be that of a typical commercial or hospital environment. If the user of the <u>MBPN002</u> and <u>MB0500300</u> requires continued operation during power mains interruptions, it is recommended that the <u>MBPN002</u> and <u>MB0500300</u> be powered from an uninterruptible power supply or a battery.
Power frequency(50, 60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	The <u>MBPN002</u> and <u>B0500300</u> power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
NOTE UT is the a.c. mains voltage prior to application of the test level.			

Technical Data

Guidance and manufacturer's declaration-electromagnetic immunity

The MBPN002 and MB0500300 are intended for use in the electromagnetic environment specified below. The customer or the user of the MBPN002 and MB0500300 should assure that is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment-guidance
Conducted RF IEC 61000-4-6	3 Vrms 150 KHz to 80 MHz	3 Vrms	<p>Portable and mobile RF communications equipment should be used no closer to any part of the <u>MBPN002</u> and <u>MB0500300</u> including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p>Recommended separation distance:</p> $d = 1,2 \sqrt{P}$ $d = 1,2 \sqrt{P} \text{ 80MHz to 800 MHz}$ $d = 2,3 \sqrt{P} \text{ 800MHz to 2,5 GHz}$ <p>Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m).</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey^a, should be less than the compliance level in each frequency range^b.</p> <p>Interference may occur in the vicinity of equipment marked with the following symbol:</p> 
Radiated RF IEC 61000-4-3	20 V/m 80MHz to 2,5 GHz	20 V/m	
<p>NOTE1 At 80 MHz and 800 MHz, the higher frequency range applies.</p> <p>NOTE2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.</p>			
<p>a. Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the <u>MBPN002</u> and <u>MB0500300</u> is used exceeds the applicable RF compliance level above, the <u>MBPN002</u> and <u>MB0500300</u> should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the <u>MBPN002</u> and <u>MB0500300</u>.</p> <p>b. Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.</p>			

Technical Data

Recommended separation distance between portable and mobile RF communications equipment and the MBPN002 and MB0500300

The MBPN002 and MB0500300 are intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the MBPN002 and MB0500300 can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the MBPN002 and MB0500300 as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter m		
	150 kHz to 80 MHz $d = 1,2 \sqrt{P}$	80 MHz to 800 MHz $d = 1,2 \sqrt{P}$	800 MHz to 2,5 GHz $d = 2,3 \sqrt{P}$
0,01	0,12	0,12	0,23
0,1	0,38	0,38	0,73
1	1,2	1,2	2,3
10	3,8	3,8	7,3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where p is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

WEEE (Directive on Wasted Electrical and Electronic Equipment)


This marking shown on the product indicates that it should not be disposed of, with other household wastes at the end of its working life. To prevent possible harm to the environment or human health from uncontrolled waste disposal, please separate this from other types of wastes and recycle it responsibly to promote the sustainable reuse of material resources.

Household users should contact either the retailer where they purchased this product, or their local government office, for details of where and how they can take this equipment for safe recycling.

Optional Parts and Accessories

Medication cup

MBPN00201

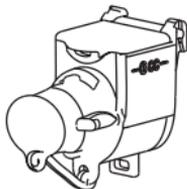Mask (Adult)

MBPN00203


Mask (Child)

MBPN00204

Power adapter

MBPN00205

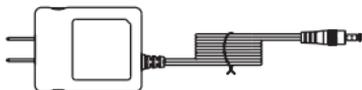- To order replacement parts or accessories, please contact your local supplier.
- The Medication cup and masks are capable of both MBPN002 and MB0500300.

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MicroBase Technology Corp.