Der richtige Weg ...
The right way ...

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*Urinary Drainage and
Incontinence Programme*

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The right way for an optimum care of urinary drainage is quite hard to overview due to the available variety of products. This brochure will show you ways of support for the patients who are at the centre of our considerations, without ignoring economic efficiency. Moreover we included important information about materials, legal foundation for Germany and a part of our comprehensive urological programme.

Editorial/ Wichtige Informationen <i>Editorial/ Important Information</i>	4
Transurethrale Ballon-Katheter <i>Transurethral Balloon Catheter</i>	6
Katheterisierungs-Bestecke <i>Catheterization Sets</i>	12
Dauerspül-Ballon-Katheter <i>Continuous Irrigation Balloon Catheter</i>	14
Suprapubische Ballon-Katheter <i>Suprapubic Balloon Catheter</i>	15
Füllmedium zum Blocken von Ballon-Kathetern <i>Filling Medium for inflating Balloon Catheter</i>	17
Katheterventile <i>Catheter Valves</i>	18
Urin-Drainage-Systeme / Urinbeutel <i>Urine Drainage Systems / Urine Bags</i>	20
Befestigung für Urin-Beinbeutel <i>Fixation Devices for Urine Leg Bags</i>	27
»SIMPLYCATH®« Einmal-Katheter und Systeme <i>»SIMPLYCATH®« Disposable Catheter and Systems</i>	30
Kondom-Urinale <i>Male External Catheter</i>	34
Verordnungstipps / Rechtsgrundlagen <i>Tips for legal Foundation in Germany</i>	35

Charrière	Millimeter
6	2,0
8	2,7
10	3,3
12	4,0
14	4,7
16	5,3
18	6,0
20	6,7
22	7,3
24	8,0
26	8,7

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About us

Since more than 50 years, UROMED Kurt Drews KG is a producer of high quality instruments and products for urology. It is our intention to improve the quality of life of the people concerned who have to live with such an impediment. At the development of all UROMED-products we attach great value on reliability and functionality. As we take into consideration the suggestions and wishes of our customers, we can continually optimise our services and the products. After all, only your satisfaction will suffice.

The reason why we recommend exclusively Silicone

The permanent treatment of the urinary drainage with Latex catheters was a standard for many decades. Disadvantages like migration and incrustation had to be minimized and new solutions had to be found.

The currently increasing number of allergies shows an additional difficulty. According to our conviction silicone catheters are the only alternative. The used biocompatible pure silicone stands for the required safety, shape and consistency for comfort and long-term application for economy. This is the base for our complete silicone programme.

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- elastic balloon

Disadvantages

- allergy of latex, caused by approx. 50 different protein molecules in the natural substance of latex
- low drainage-performance due to small inner lumen
- high incrustation (deposition of urine-salt), therefore short indwelling time (see rules of RKI [Robert Koch-Institut])
- migration: latex evaporates (visible fading of the catheter)
- frequent change of catheter (therefore higher expenses)

UROMED Silicone

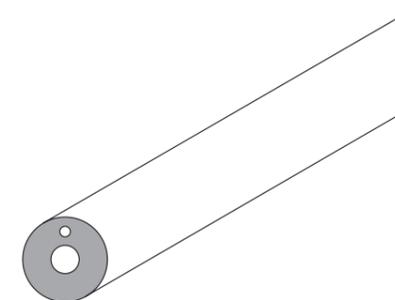
Advantages

- high drainage-performance (90% increased compared with a latex catheter of same size)
- no allergic reactions
- material scientifically accepted
- no migration
- long-term application
- low incrustation rate (deposition of urine-salt)
- soft balloon

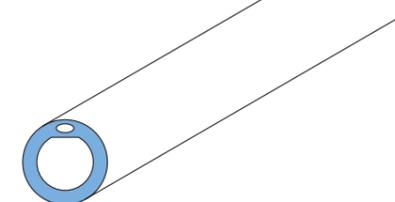
Disadvantages

- diffusion in the area of balloon (see page 17)

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Our Service

We also work with the same degree of care in the service as we devote to the selection of our suppliers and the manufacture of our products. Our responsibility does not end at the point of selling, it includes competent advice. Qualified medical device consultants place their support at our clients disposal at any time.

Our comprehensive store guarantees shortest delivery times. This service has been appreciated by our clients for a long time.

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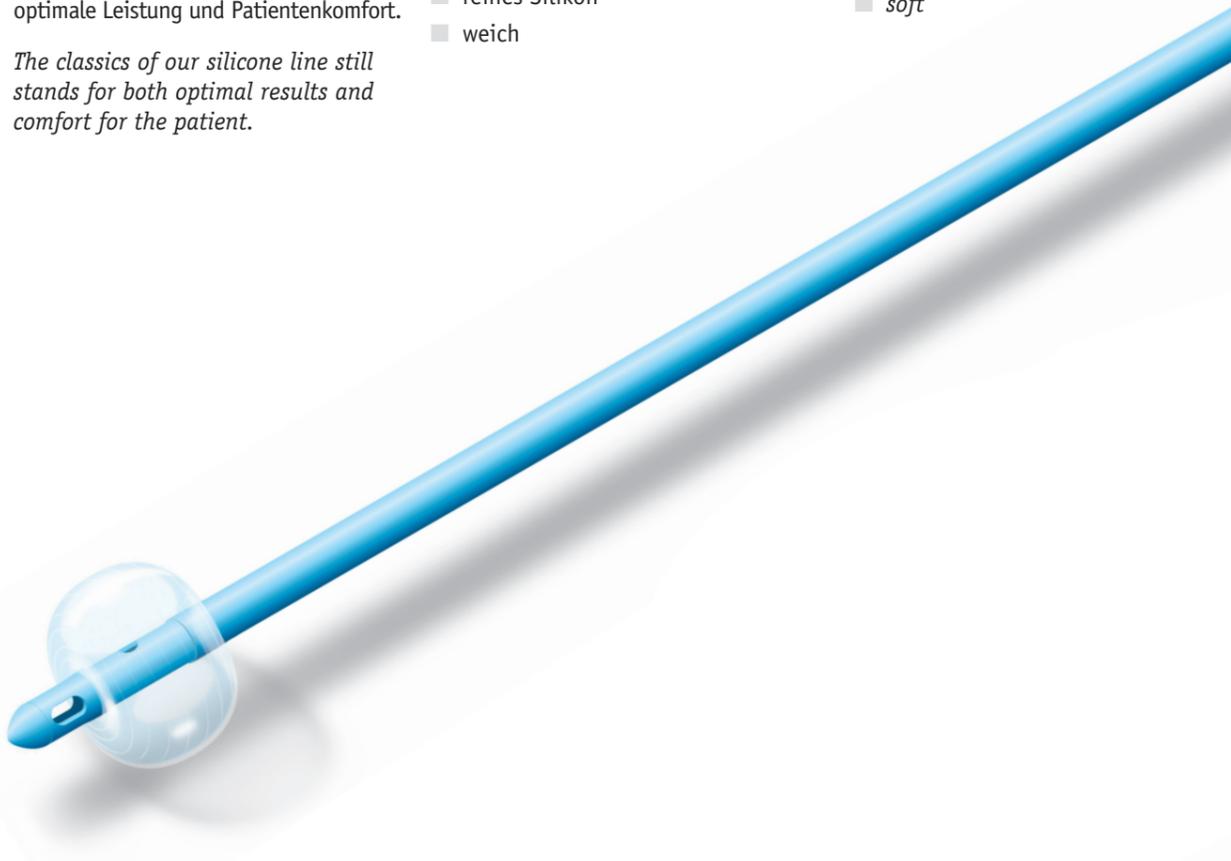
The classics of our silicone line still stands for both optimal results and comfort for the patient.

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Ballon catheter for transurethral bladder drainage

- short conical tip
- wide lumen
- radiopaque tip
- soft balloon
- pure silicone
- soft

REF REF	Größen Sizes	Ballons Balloons	PZN PZN
1100 - 12	Ch. 12	3 – 6 ml	03280661
1100 - 14	Ch. 14	3 – 8 ml	03280678
1100 - 16	Ch. 16	5 – 10 ml	03280684
1100 - 18	Ch. 18	5 – 10 ml	03280690
1100 - 20	Ch. 20	5 – 10 ml	03280709
1100 - 22	Ch. 22	5 – 10 ml	03280715
1100 - 24	Ch. 24	5 – 10 ml	03280721
1100 - 26	Ch. 26	5 – 10 ml	03280738

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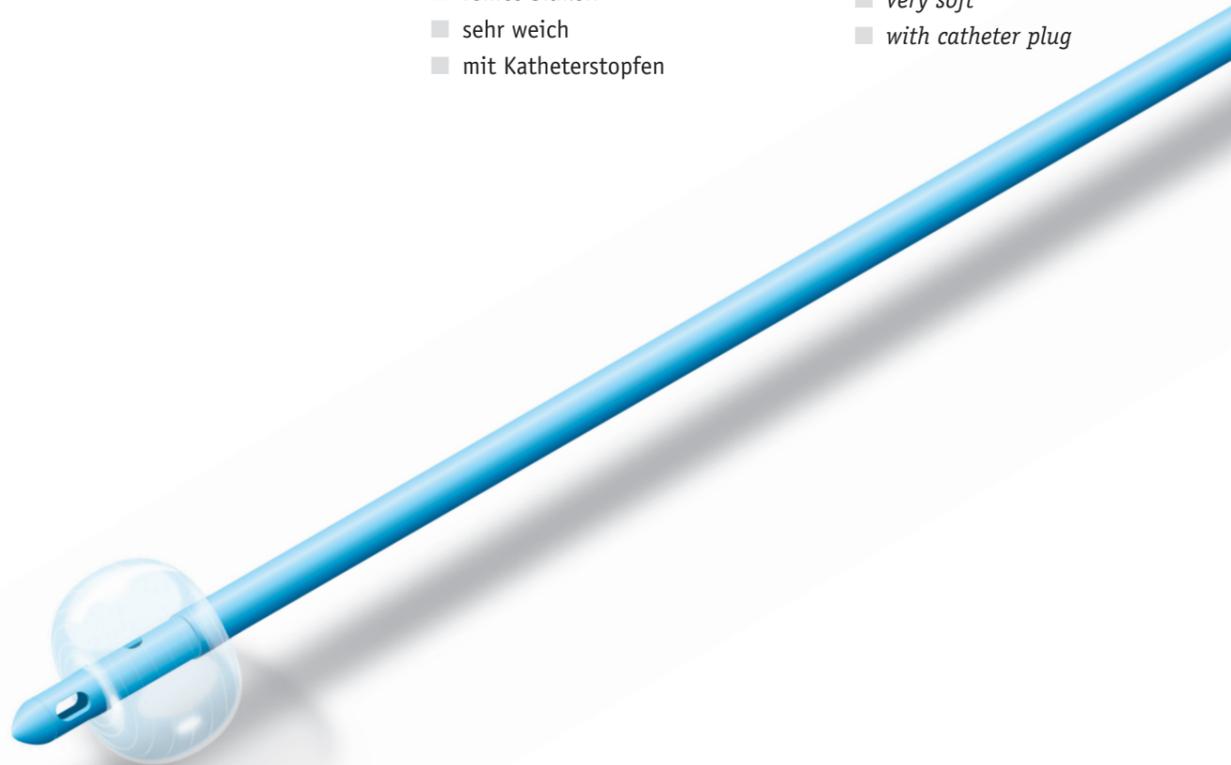
Ballon catheter for transurethral bladder drainage

- short, conical tip
- wide lumen
- radiopaque tip
- soft balloon
- pure silicone
- very soft
- with catheter plug

REF REF	Größen Sizes	Ballons Balloons	PZN PZN
1095-12	Ch. 12	3- 6 ml	04741526
1095-14	Ch. 14	3- 8 ml	04741532
1095-16	Ch. 16	5- 10 ml	04741549
1095-18	Ch. 18	5- 10 ml	04741555
1095-20	Ch. 20	5- 10 ml	04741561
1095-22	Ch. 22	5- 10 ml	04741578

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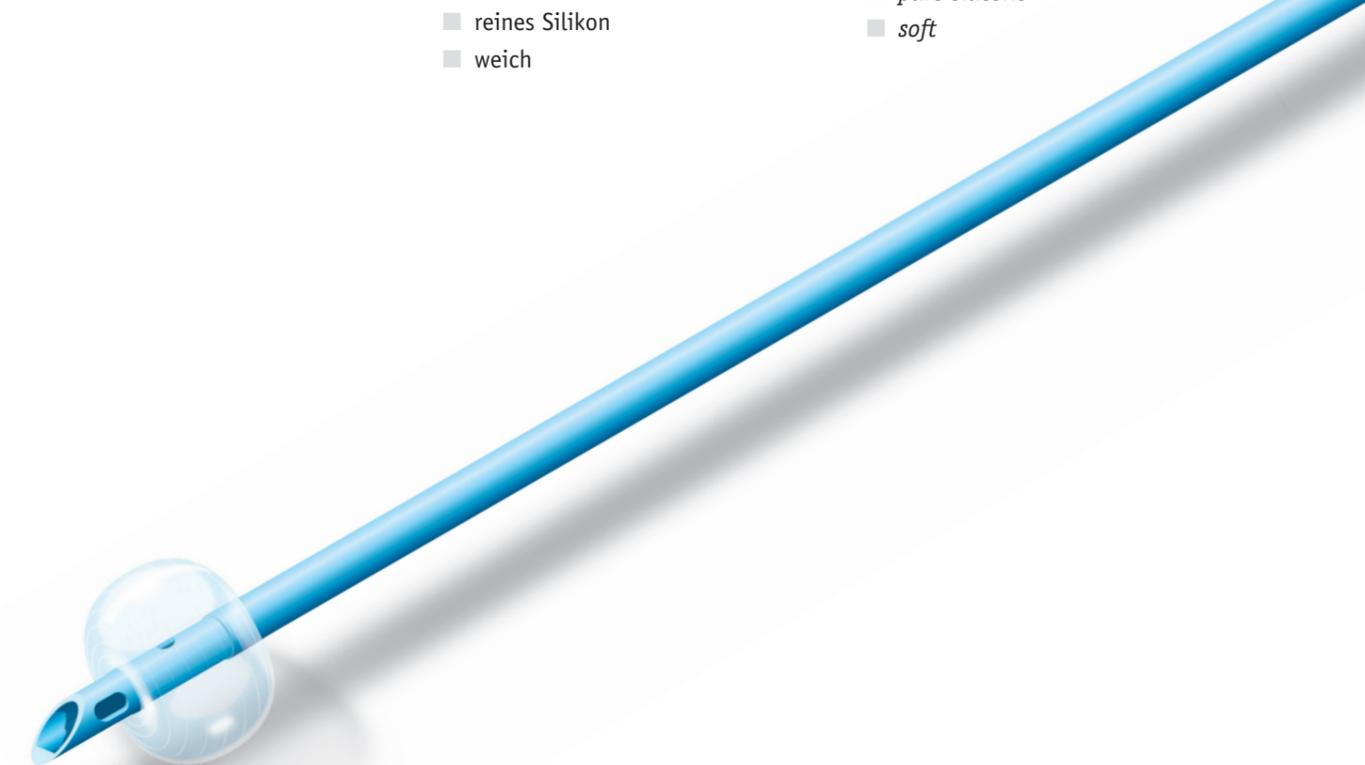
Ballon catheter for transurethral bladder drainage

This catheter allows a safe drainage even for problematic urinary findings.

- open, short tip
- wide lumen
- soft balloon
- pure silicone
- soft

REF REF	Größen Sizes	Ballons Balloons	PZN PZN
1091-12	Ch. 12	3- 6 ml	03953657
1091-14	Ch. 14	3- 8 ml	03953692
1091-16	Ch. 16	5- 10 ml	03953700
1091-18	Ch. 18	5- 10 ml	03953717
1091-20	Ch. 20	5- 10 ml	03953723

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- offene Spitze
- großlumig
- Soft-Ballon
- reines Silikon
- weich

Ballon catheter for transurethral bladder drainage

This catheter allows a safe drainage even for problematic urinary findings.

- *open tip*
- *wide lumen*
- *soft balloon*
- *pure silicone*
- *soft*

REF REF	Größen Sizes	Ballon Balloon	PZN PZN
1090-16	Ch. 16	30 ml	04741495
1090-18	Ch. 18	30 ml	04741503
1090-20	Ch. 20	30 ml	03280649
1090-22	Ch. 22	30 ml	03280655

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Ballon catheter for transurethral bladder drainage for women

- *short, conical tip*
- *wide lumen*
- *radiopaque tip*
- *soft balloon*
- *pure silicone*
- *soft*

REF REF	Größen Sizes	Ballons Balloons	PZN PZN
1370-12	Ch. 12	3- 8 ml	03280862
1370-14	Ch. 14	3- 8 ml	03280879
1370-16	Ch. 16	5- 10 ml	03280885
1370-18	Ch. 18	5- 10 ml	03280891
1370-20	Ch. 20	5- 10 ml	03280916
1370-22	Ch. 22	5- 10 ml	03280922
1370-24	Ch. 24	5- 10 ml	03280939

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The UROMED catheterization sets ensure that all accessories for a hygienic catheter change will be at your disposal.

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Set for the change of urological catheter

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Set for the change of urological balloon catheter

same as catheterization set »PLUS« REF 1565 but additionally with UROMED filling medium REF 1550.10

■ especially for silicone balloon catheter

REF	PZN
REF	PZN
1566	00959464

Verpackung: steril
Original-Karton: 40 Stück
Packing: sterile
Original carton: 40 pieces

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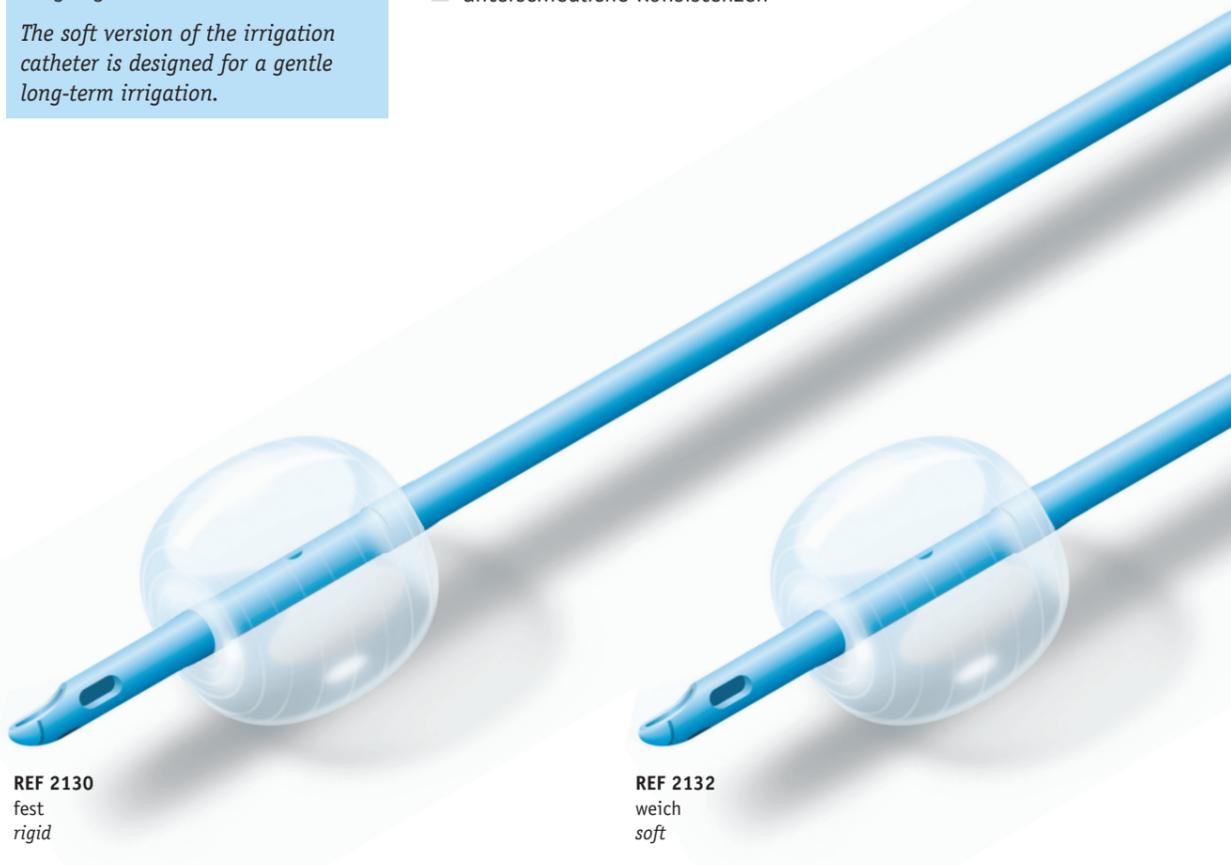
The soft version of the irrigation catheter is designed for a gentle long-term irrigation.

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Continuous irrigation balloon catheter for postoperative bladder drainage

- short, non-kinking tip
- high irrigation and drainage results
- radiopaque tip
- large, staggered eyes
- pure silicone
- different consistencies

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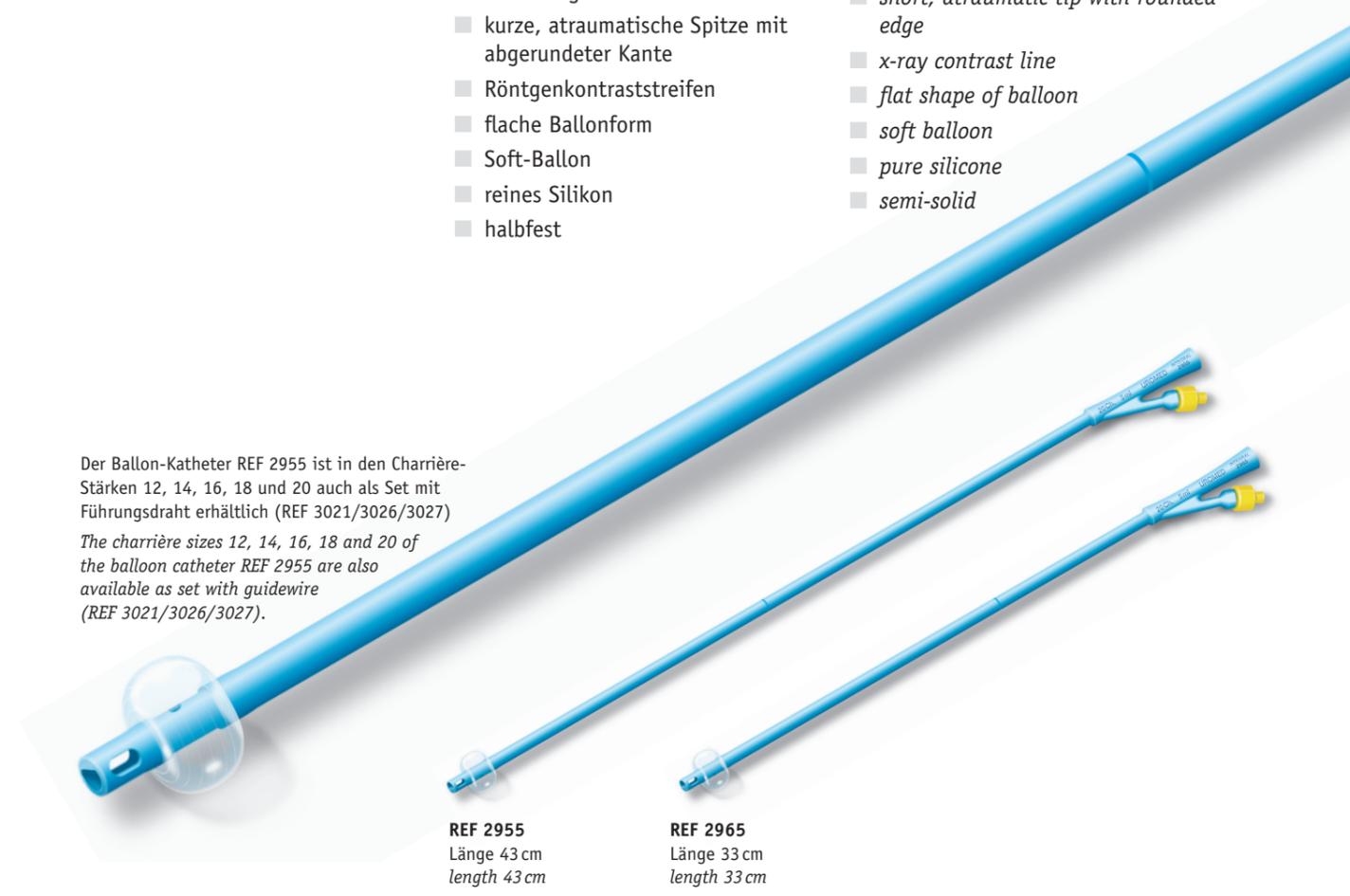
Balloon catheter for suprapubic drainage of the urinary bladder

- »INTEGRAL« balloon catheter with integrated flat balloon allow to a great extent a painless change of the catheter and make it easier to pass the fascie.
- short, atraumatic tip with rounded edge
- x-ray contrast line
- flat shape of balloon
- soft balloon
- pure silicone
- semi-solid

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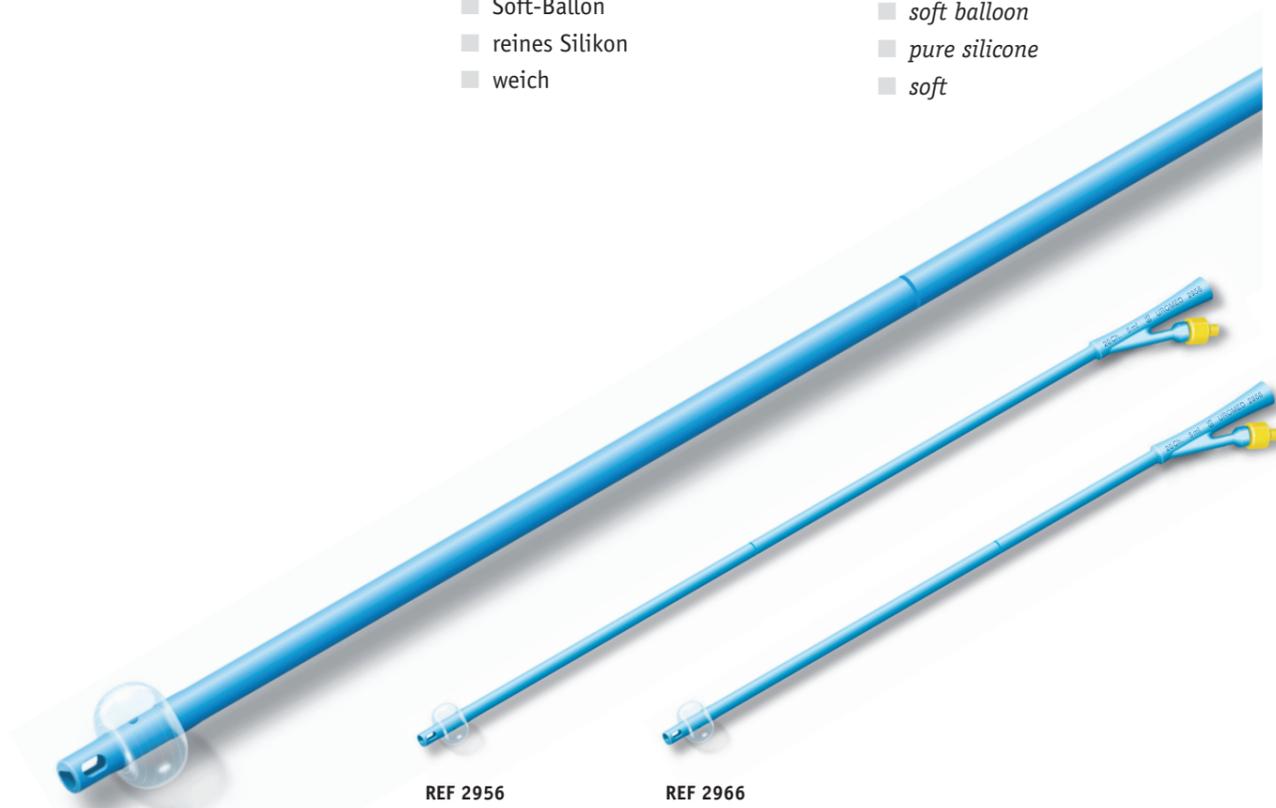
Ballon catheter for suprapubic drainage of the urinary bladder

- short, atraumatic tip with rounded edge
- flat shape of balloon
- overall radiopaque
- wide lumen
- soft balloon
- pure silicone
- soft

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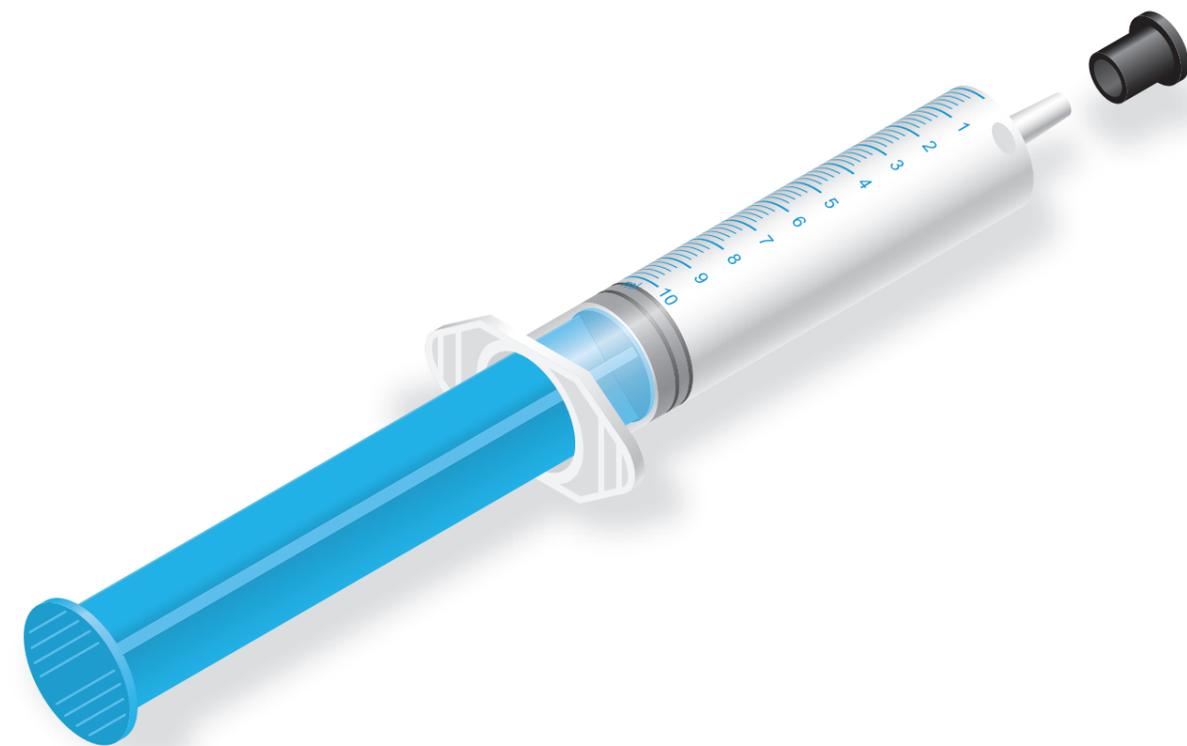
Luer syringe filling medium for inflating urological balloon catheter

- contains 10% glycerine in aqua dest.
- keeps the filling volume constant
- easier application of catheter
- free of latex

REF REF	Volumen Volume	PZN PZN
1550-05	5 ml	00730833
1550-10	10 ml	00730827

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Diffusion by silicone balloon catheter

The balloons of the silicone catheter in general allow a diffusion caused by the material. Therefore the balloon probably might evacuate slowly. This applies especially the nephrostomy and suprapubic balloon catheter with small filling volume. The UROMED filling medium helps to reduce any diffusion effectively due to the glycerin.

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To urinate at the touch of a button. This means use free of contamination and methodical bladder training.

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Catheter valve to control the emptying of the urinary bladder

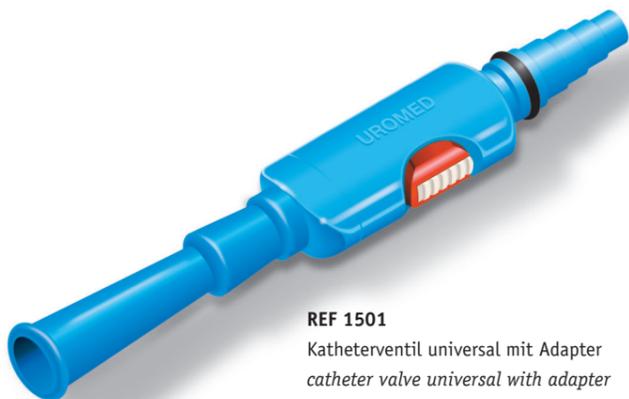
- any contamination due to repeated insertion and removal of a catheter plug will be avoided
- one-handed use
- smooth pressing mechanism
- wide lumen for fast emptying
- automatic closing after release of the push button
- optimal drop separation
- very light (10 g)
- contoured shape
- permanent opening with integrated pusher
- safe and fast adapting of a CYSTOBAG urine bag without manipulating the catheter

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The UROMED catheter valve compact should be used if the connection with an urine bag is not required. It does not offer a mechanism for permanent opening.

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Catheter valve to control the emptying of the urinary bladder

- any contamination due to repeated insertion and removal of a catheter plug will be avoided
- one-handed use
- smooth pressing mechanism
- wide lumen for fast emptying
- automatic closing after release of the push button
- optimal drop separation
- very light (10 g)
- contoured shape

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Bladder training with the UROMED catheter valve

The constant drainage of urine into an urine bag leads to a permanent emptying of the bladder. The bladder musculature languishes as the natural rhythm of emptying is interrupted. With the UROMED catheter valve the patient can counteract methodically because the urination will happen after filling of the bladder only.

REF	PZN
REF	PZN
1504	00051888

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CYSTOBAG® urine drainage systems are based on the actual needs of the users in practice – go and find out for yourself!

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Urine bag for collection of extracted urine

- with gloves free of latex
- needleless puncture spot for urine specimen
- tube clamp
- clamp for tube fixation
- two-part drip chamber with recoil blockage
- air-filter on drip chamber and bag
- twin holder with fixing cord for all-purpose use
- clearly graduated
- T-tap-port
- T-tap-outlet, one-handed use without contamination
- tap with wide lumen for fast emptying
- extremely short bag, 29 cm
- label for patient data

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A tube with profile.

The objective is to maintain a drainage at all events. The octagonally profiled supply tube guarantees a drainage even if it is bent or difficult to position the patients correctly.

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The All-Purpose-Holder.

The UROMED twin holder stands out because it can be used on various bedsteads and always gets into a vertical position. A free fall of the drop is ensured and an ascent of germs will be avoided.

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Step connector for reliable links.

With the distinct steps of the connector an optimal link with the catheter will be achieved and leakage will be avoided. Due to the direct adjustment to the funnels of the UROMED silicone balloon catheter the link is particularly good in this case.

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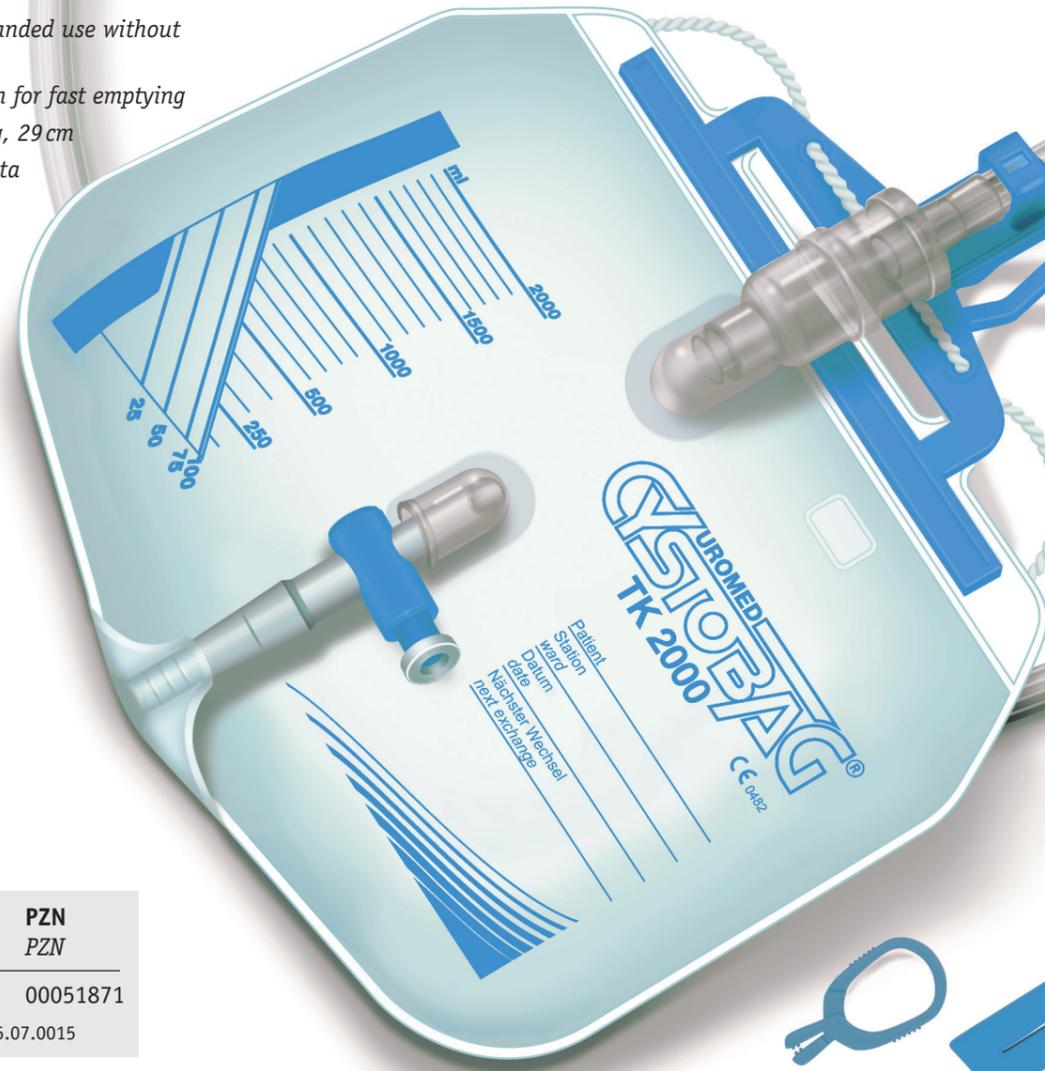
Easy emptying.

The t-tap-outlet for one-handed use without contamination ensures a fast drain due to its wide inner diameter.

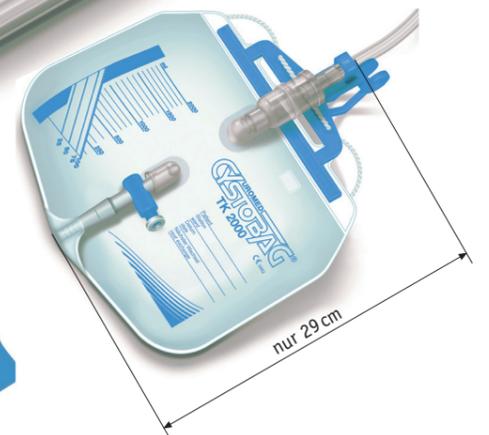
REF	Volumen	PZN
REF	Volume	PZN
4802	2000 ml	00051871

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Urine bag for collection of extracted urine

- flexible tube
- needleless puncture spot for urine specimen
- tube clamp
- non-return valve
- T-tap-outlet, one-handed use without contamination
- clearly graduated
- label for patient data

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closed urine drainage system, 1000 ml, day and night care, fleece cover, sterile

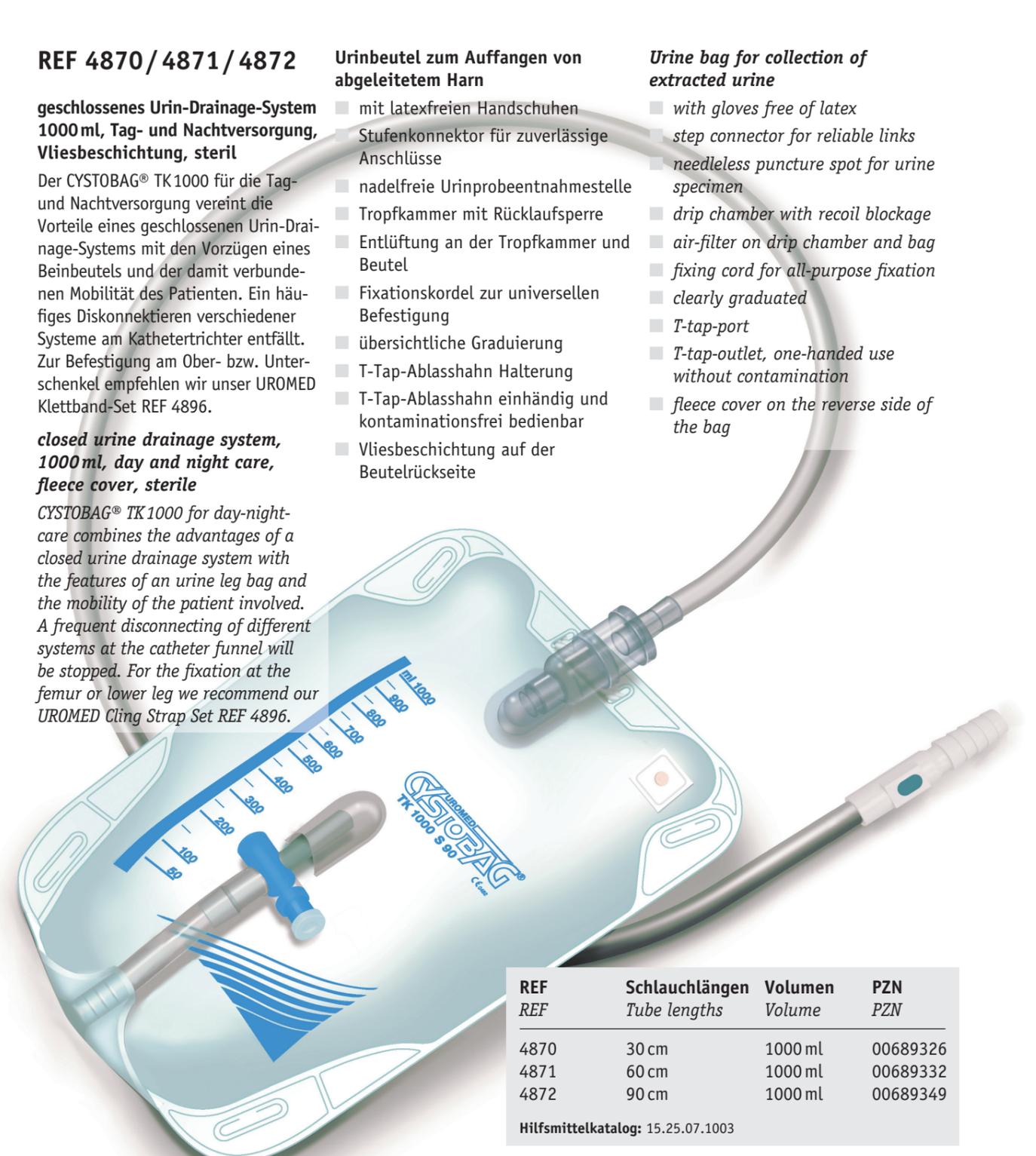
CYSTOBAG® TK 1000 for day-night-care combines the advantages of a closed urine drainage system with the features of an urine leg bag and the mobility of the patient involved. A frequent disconnecting of different systems at the catheter funnel will be stopped. For the fixation at the femur or lower leg we recommend our UROMED Cling Strap Set REF 4896.

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Urine bag for collection of extracted urine

- with gloves free of latex
- step connector for reliable links
- needleless puncture spot for urine specimen
- drip chamber with recoil blockage
- air-filter on drip chamber and bag
- fixing cord for all-purpose fixation
- clearly graduated
- T-tap-port
- T-tap-outlet, one-handed use without contamination
- fleece cover on the reverse side of the bag

REF	Schlauchlängen	Volumen	PZN
REF	Tube lengths	Volume	PZN
4870	30 cm	1000 ml	00689326
4871	60 cm	1000 ml	00689332
4872	90 cm	1000 ml	00689349
Hilfsmittelkatalog: 15.25.07.1003			

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Packing: sterile
Original packing: 20 pieces

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Urine bag with fleece cover, non sterile

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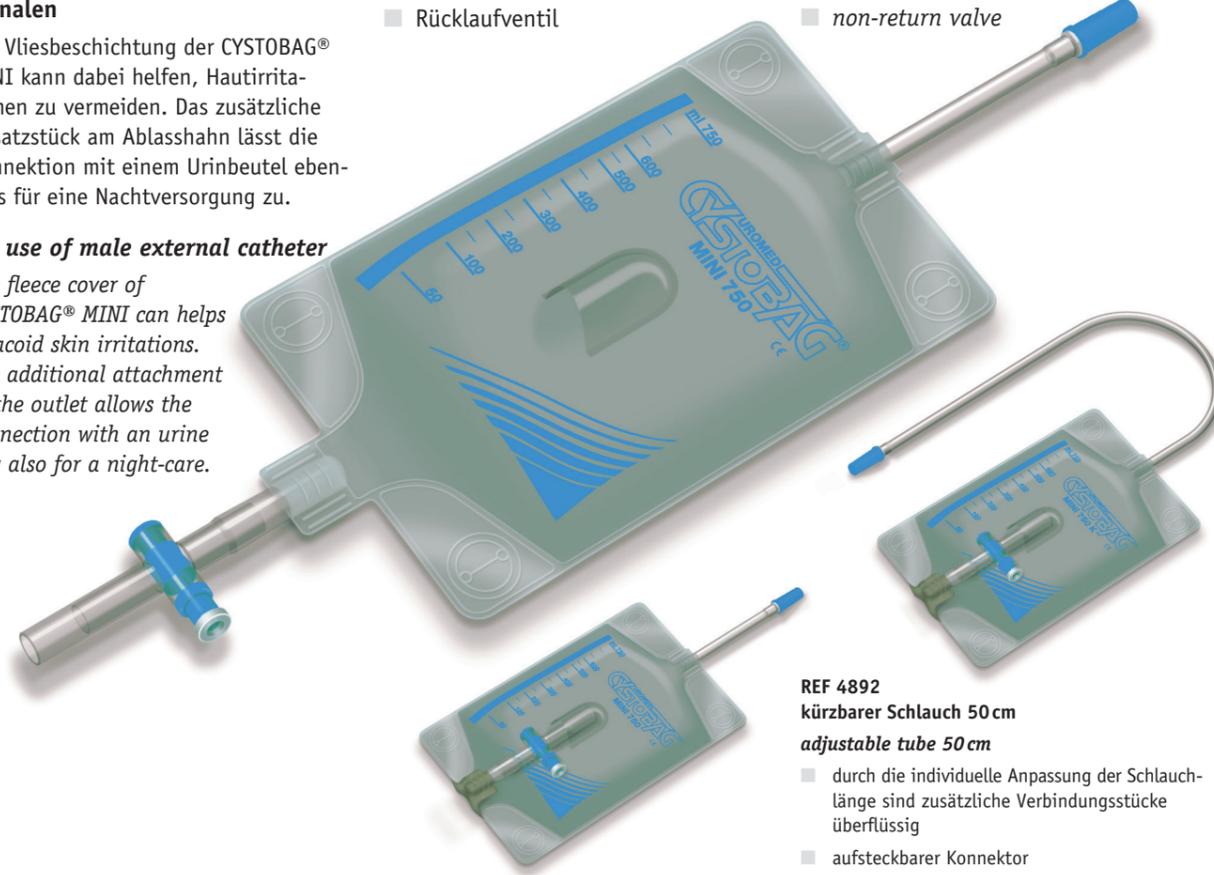
Urine bag for collection of extracted urine

- short bag length due to tip-up T-tap-outlet
- T-tap-outlet, one-handed use without contamination
- attachment for night bags
- non-return valve

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urine bag with fleece cover, sterile

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for patients with permanent catheter

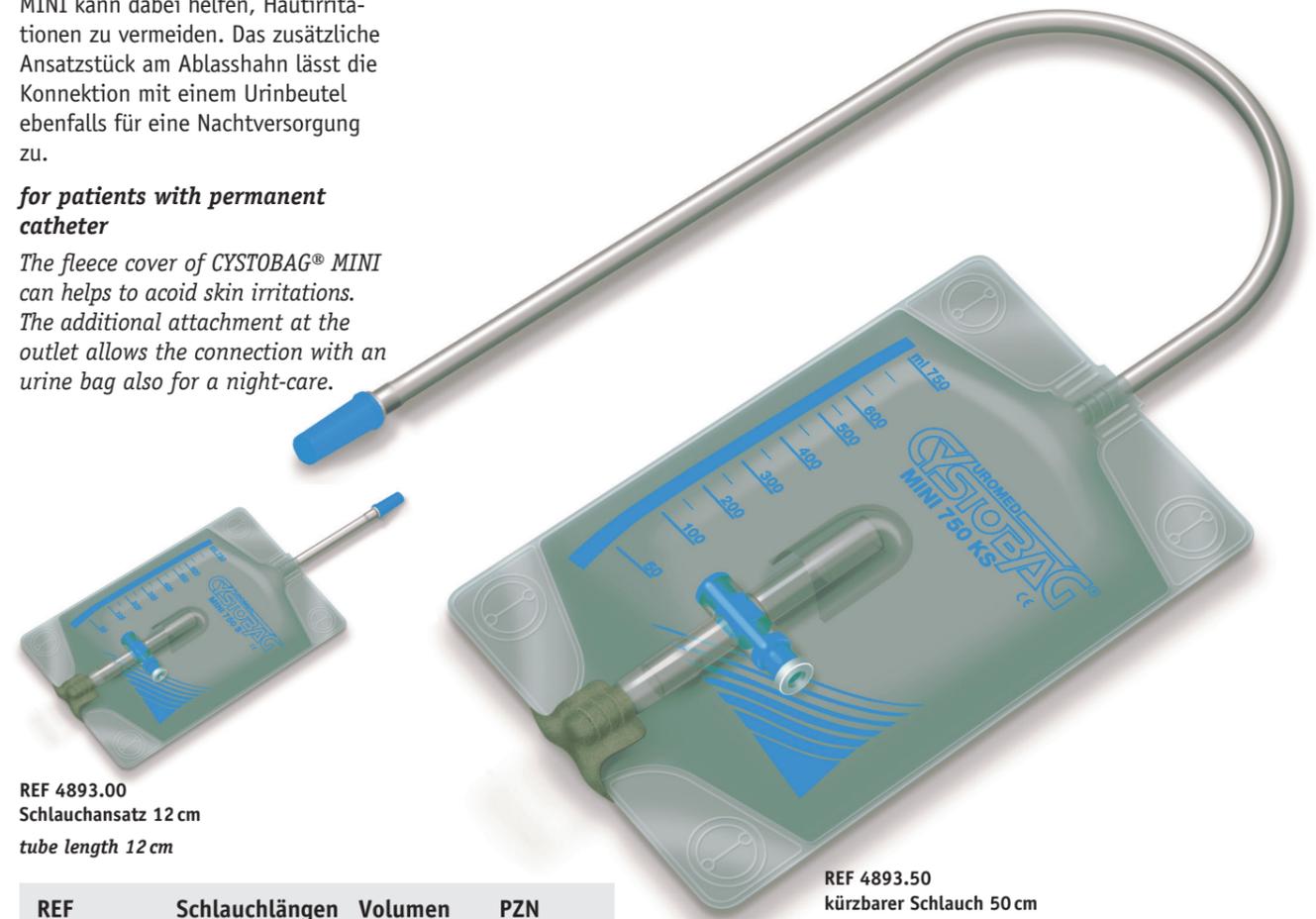
The fleece cover of CYSTOBAG® MINI can help to avoid skin irritations. The additional attachment at the outlet allows the connection with an urine bag also for a night-care.

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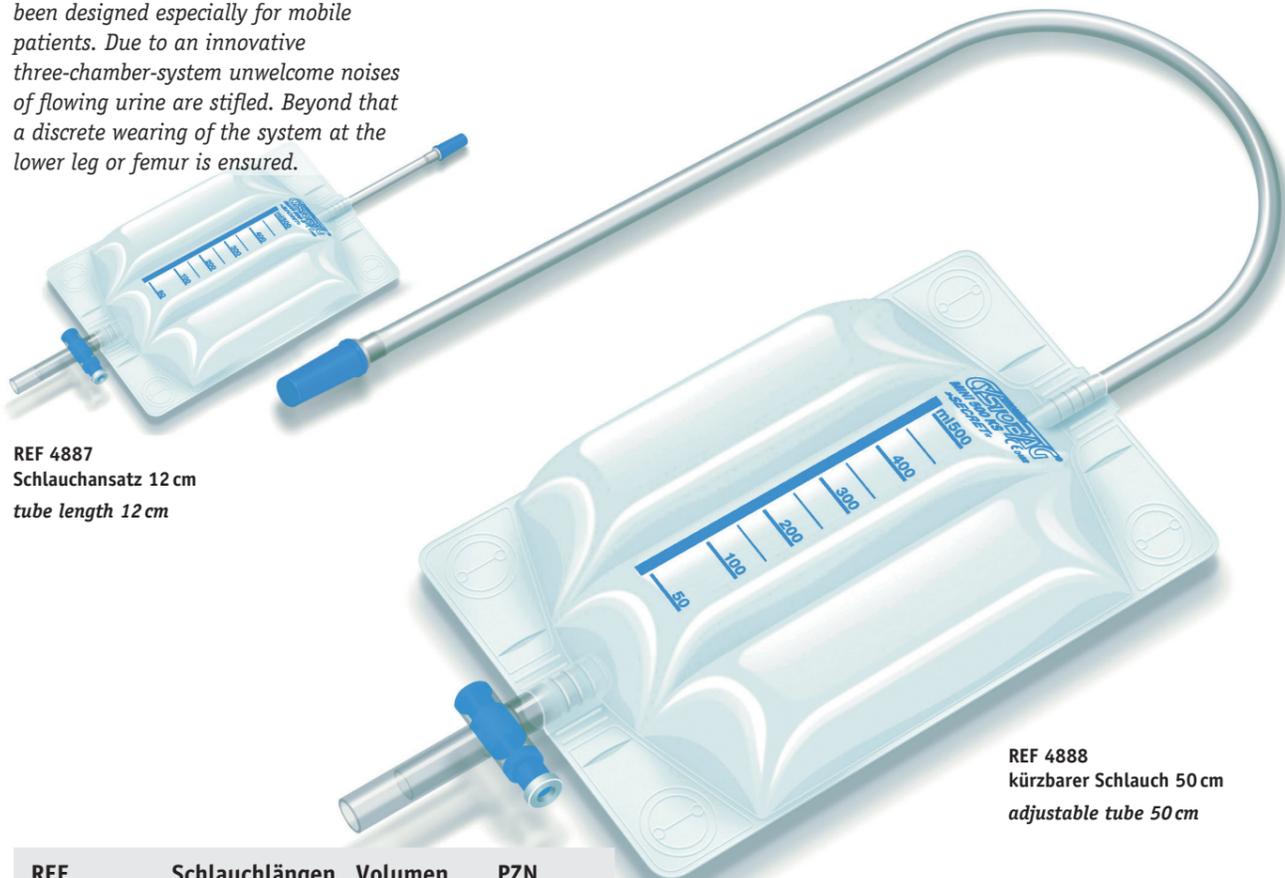
CYSTOBAG®-MINI 500 »SECRET« has been designed especially for mobile patients. Due to an innovative three-chamber-system unwelcome noises of flowing urine are stifled. Beyond that a discrete wearing of the system at the lower leg or femur is ensured.

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Urine bag for collection of extracted urine

- short bag length
- T-tap-outlet, one-handed use without contamination
- attachment for night bags
- non-return-valve
- fleece cover on the reverse side of the bag

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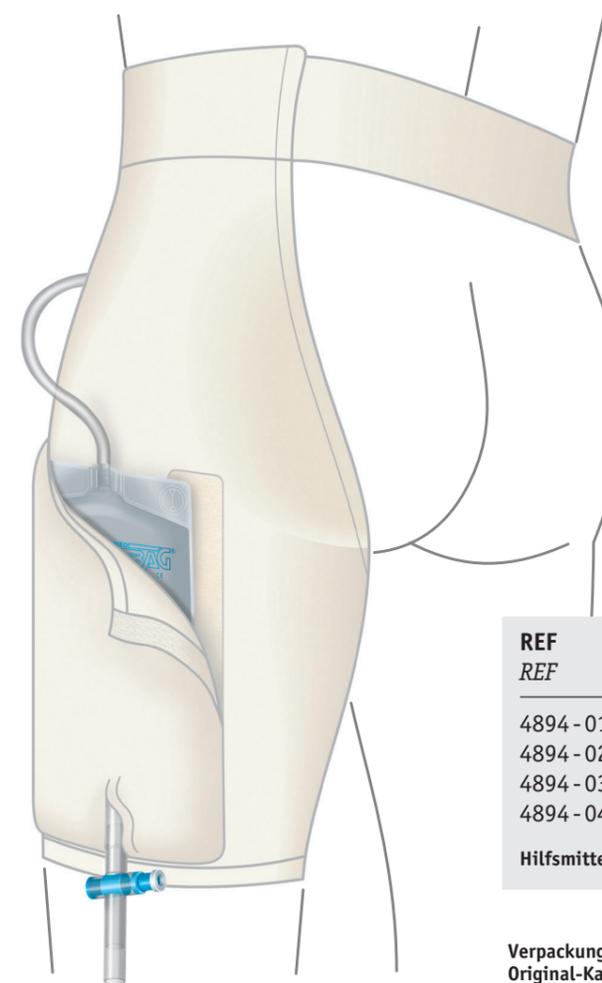
Make it easier for your mobile patients to wear a leg bag. The construction of the UROMED one-leg-pant makes it possible to wear it at the right leg as well as at the left leg. A cling strap fastening simplifies the insertion of the urine bag into the pocket.

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One-leg-pant for CYSTOBAG® urine leg bags

- pleasant to wear
- more freedom of movement
- no slipping of the bag
- easy emptying of the urine bag
- for unhindered blood circulation
- biologically examined pre-washed cotton mixed fabric (free of latex)
- wide elastic belt
- simple and individually adjustable cling strap fastening

REF REF	Größen Sizes	Taillenumfang Waist circumference	PZN PZN
4894-01	»S« klein/small	79 – 87 cm	04743583
4894-02	»M« mittel/medium	85 – 95 cm	04743608
4894-03	»L« groß/large	90 – 100 cm	04743614
4894-04	»XL« extra groß/extra large	98 – 120 cm	04743620

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The UROMED leg bag holder permits the unremarkable wear of urine bags and offers a sufficient fixation by elastic, wide cuff.

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Leg bag holder for CYSTOBAG® urine leg bags

- with an integrated pocket
- pleasant to wear
- more freedom of movement
- no slipping of the bag
- no direct contact between skin and bag
- easy emptying of the urine bag
- latex-free mixed fabric (80% polyester, 20% elastane)
- wearable either at the right or left upper/lower leg

REF REF	Größen Sizes	Beinumfang Leg circumference	PZN PZN
4897-01	»S« klein/small	30 – 45 cm	09704091
4897-02	»M« mittel/medium	45 – 65 cm	09704116
4897-03	»L« groß/large	55 – 80 cm	09704139
4897-04	»XL« extra-groß/extra large	65 – 90 cm	09704145

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The upper and crucial fixation of this cling strap set is the wide and comfortable cling strap. The lower fixation strap may be adjusted individually according to patient's needs.

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Cling strap set for fixation of CYSTOBAG® urine leg bags

- comfortable and economic because all components are offered in the set
- no slipping of the bag
- safe fixation with silicone cover
- elastic and eudermic material
- not constricting
- free of latex
- big flap (D-ring) for easy opening

REF REF	Größen Sizes	Beinumfang Leg circumference	PZN PZN
4896-01	»S« klein/small	23 – 33 cm	03548974
4896-02	»M« mittel/medium	33 – 51 cm	03548980
4896-03	»L« groß/large	51 – 68 cm	03549502

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Straps for fixation of CYSTOBAG® urine leg bags

- safe fixation with silicone cover
- elastic and eudermic material
- not constricting
- can be adjusted
- free of latex

REF REF	PZN PZN
4895	00051836

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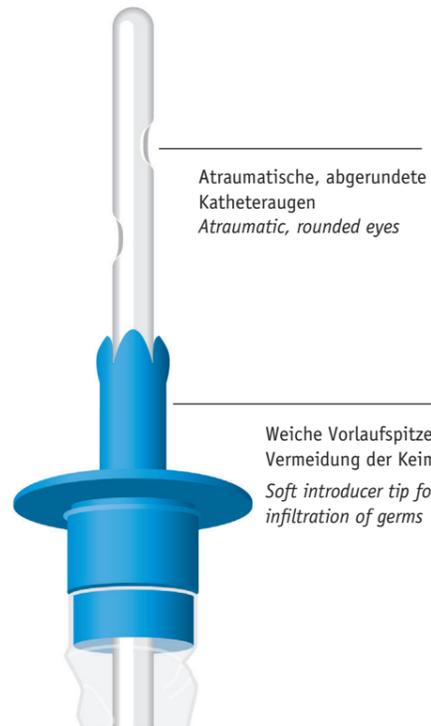
Disposable catheter for intermittent self-catheterization

- introducer tip for preventing the infiltration of germs
- gel is evenly spread over the entire catheter
- atraumatic eyes
- immediately usable
- easy to use
- thin foil with anti-slip-effect
- small in size, discrete in use
- opaque, low-noise packaging
- sophisticated design
- DEHP-free materials

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Angenehm weiches Beutelmateriale in Flaschenform
Comfortable and soft bag material in bottle shape

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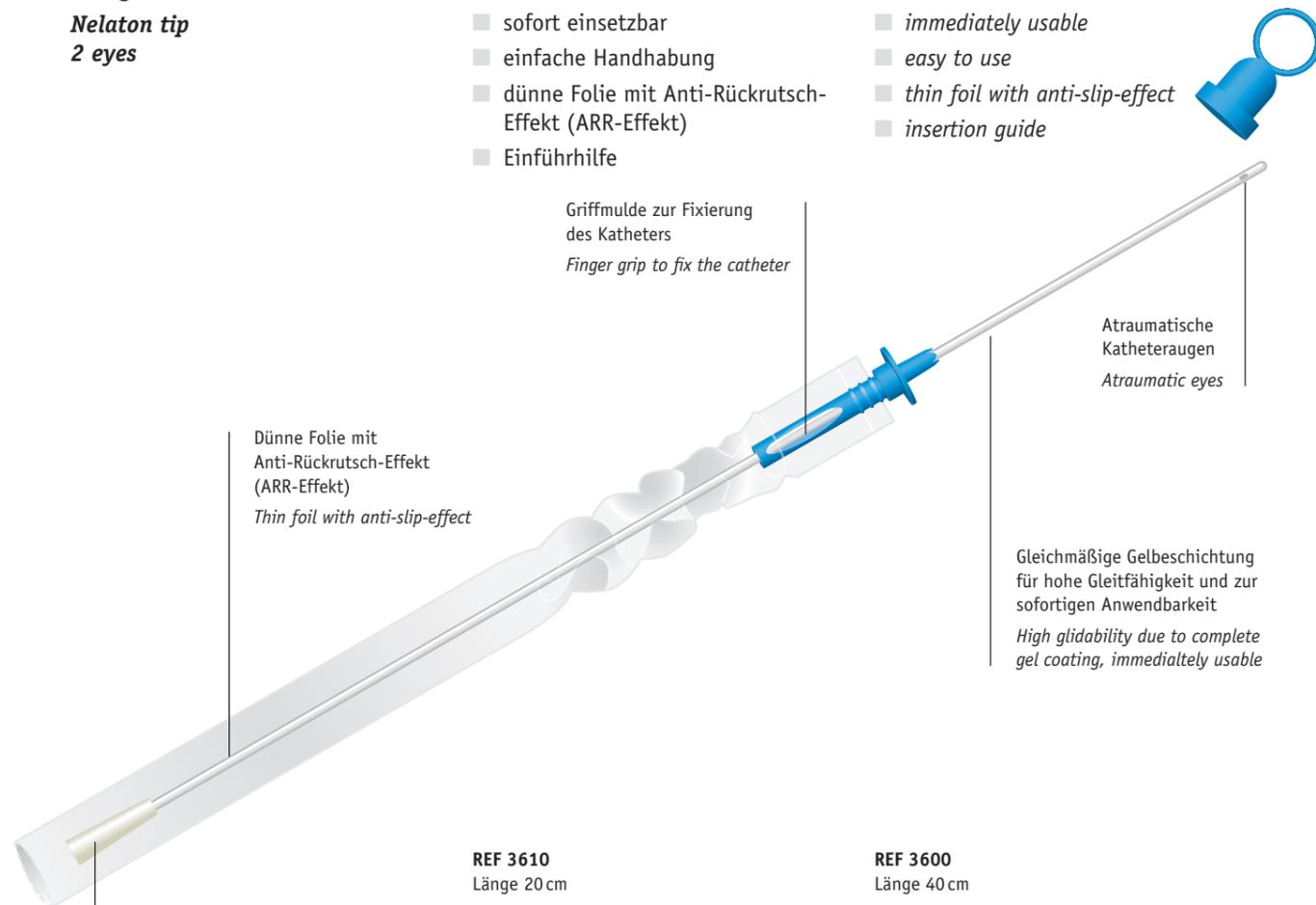
- *introducer tip for preventing the infiltration of germs*
- *gel is evenly spread over the entire catheter*
- *atraumatic eyes*
- *immediately usable*
- *easy to use*
- *thin foil with anti-slip-effect*
- *insertion guide*

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Hilfsmittelkatalog: 15.25.14.5058

Packing: sterile
Original packing: 50 pieces

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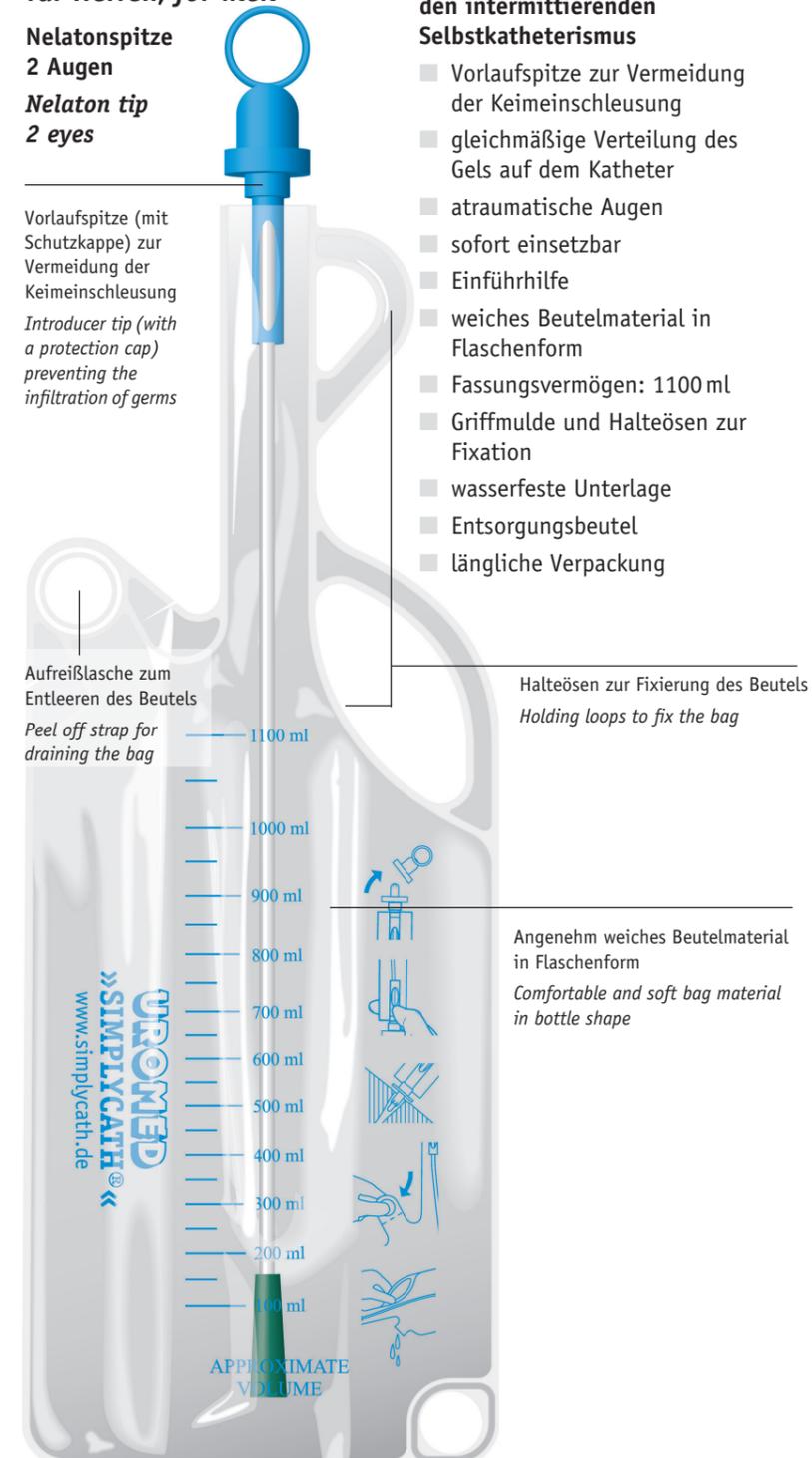
Disposable catheter system with connected urine bag for intermittent self-catheterization

- *introducer tip for preventing the infiltration of germs*
- *gel is evenly spread over the entire catheter*
- *atraumatic eyes*
- *immediately usable*
- *insertion guide*
- *soft material of the bag in bottle shape*
- *capacity: 1100 ml*
- *finger grip and holding loops for fixation*
- *waterproof bolster*
- *waste bag*
- *elongate package*

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*Optimal for using on the way.
The packaging includes:*

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Packing: sterile
Original packing: 30 pieces

UROMED SILIKON KONDOM-URINALE

UROMED SILICONE MALE EXTERNAL CATHETERS

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Self-adhesive male external catheter

The self-adhesive silicone male external catheters are non-allergic, breathable and easy to use.

The skin-friendly adhesive coating is providing the highest possible safe and reliable adhesion. All designs are made of transparent, pure silicone.

REF 4940

Male external catheter STANDARD

- Short external catheter for use on a retracted penis
- Adhesive surface is optimised for anatomical conditions
- Secure connection with the most common urine bags
- Rolls down very easily

REF REF	Größen Sizes	PZN CPN
4940-24	20-25 mm	13650159
4940-28	25-30 mm	13650165
4940-31	30-35 mm	13650171

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REF 4960

Male external catheter SPORTIV

- Large adhesive surface for maximum safety and reliability for sporty activities or active lifestyle
- Ideal solution even for wheelchair users thanks to the optimised adhesive surface and the reservoir
- Secure connection with the most common urine bags
- Rolls down very easily

REF REF	Größen Sizes	PZN CPN
4960-24	20-25 mm	13650188
4960-28	25-30 mm	13650194
4960-31	30-35 mm	13650202
4960-35	35-40 mm	13650219

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Packing: individually packaged in a hygienic polybag, unsterile
Original carton: 30 pieces

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The above description regarding the prescription of our products is destined for Germany only and, for this reason, not translated into the English language.

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The represented products are only an extract of our comprehensive urological programme.

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All indications correspond to our knowledge at the date of press. Due to technical progress all products are subject to change.

Material-Nr. 0009 · Rev. 4 · 02/2022

© UROMED Kurt Drews KG